
Производитель
MicroLife AG
Espanstrasse 139
CH-9143 Widnau / Switzerland
Tel: +41 71 777 71 30
Fax: +41 71 777 71 30
Email: admin@microlife.ch
www.microlife.com
MicroLife AT
Mocniktrgasse 139
CH-9143 Widnau / Switzerland
Tel: +41 71 777 71 30
Fax: +41 71 777 71 30
Email: admin@microlife.ch

MicroLife NEB 50

NEB 50
IB NEB 50A RU-V2 815

microlife

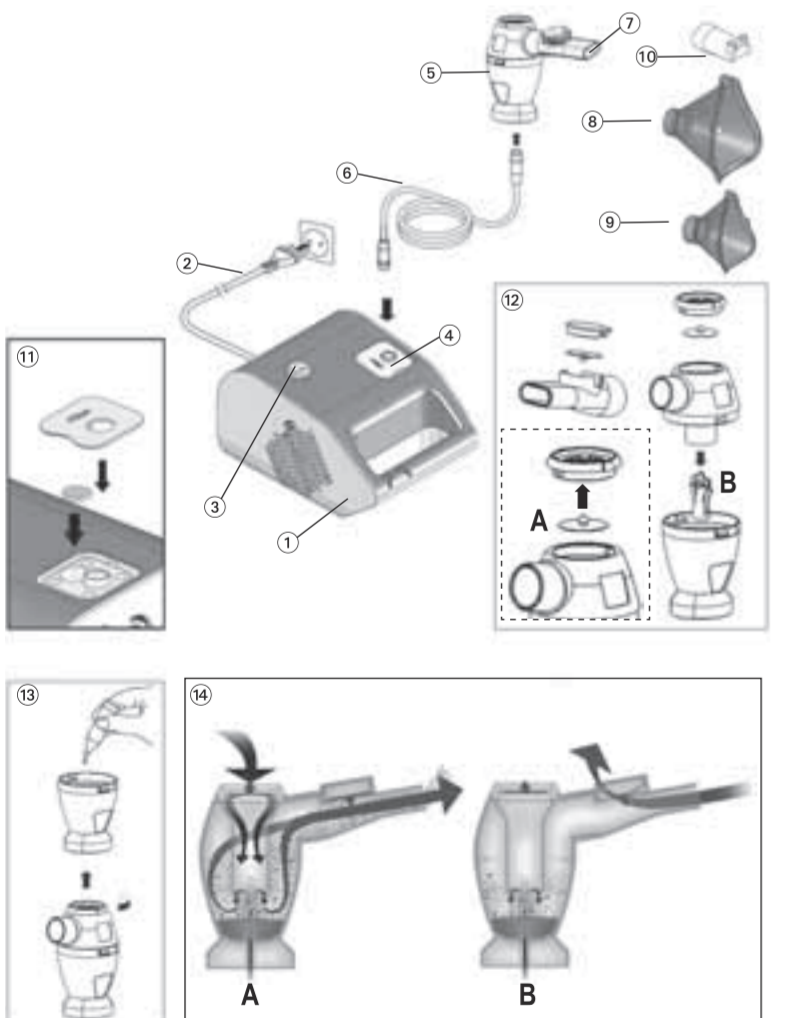CE 0434

3A Health Care S.r.l.
Via Marziale Ceruti Localita San Tommaso
25017 Lonato (BS) - Italy
Tel. 0039 030 9133177
Fax 0039 030 9919114

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Medical compressor nebuliser Model NEB 50

- ① Medical compressor nebuliser
- ② Power lead
- ③ ON/OFF Switch
- ④ Air filter compartment
- ⑤ Nebuliser
- ⑥ Air hose
- ⑦ Mouthpiece for inhalation through the mouth
- ⑧ Adult face mask
- ⑨ Child face mask
- ⑩ Nose piece for inhalation through the nose
- ⑪ Replacing air filter
- ⑫ Assembling nebuliser kit
- ⑬ Filling the nebuliser
- ⑭ Breath-enhanced nebulisation

Dear Customer,
This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – MicroLife AG!

1. **Explanation of symbols**
 - ⚡ Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
 - 📖 Read the instructions carefully before using this device.
 - 👤 Type B applied part
 - 🕒 Keep dry
 - ☑ Conformity mark
 - 📦 Class II equipment
 - 📄 LOT Lot number
 - 📄 REF Reference number
 - 🏭 Manufacturer
 - 🌡 Operating
Operation temperature restriction
10 - 40 °C
 - 🌡 Storage
Storage temperature restriction
-25 - +70 °C

CE 0434 CE Marking of Conformity

2. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never bend the nebuliser over 60°
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- ⚠ Use of this device is not intended as a substitute for a consultation with your doctor.

3. Contraindications

For type, dose, contraindication and regime of medication follow the instructions of your doctor or respiratory therapist. Please carefully read the instructions of the drug before using it.

4. Preparation and Usage of this Device

- Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».
1. Assemble the nebuliser kit (12). Ensure that all parts are complete.
 2. Fill the nebuliser (13) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
 3. Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (230V~ / 50Hz).
 4. Switch the ON/OFF switch (3) to position «I» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.
 - Use all accessories including the nose piece (10) as prescribed by your doctor.
 5. Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
 6. After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «O» to turn off the device and unplug it from the socket.
 7. Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

☞ **This device was designed for intermittent use of 30 min. On / 30 min. Off.** Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

5. Nebuliser treatment options

This device is equipped with the innovative nebuliser «NEB PRO» (5) which includes the breath-enhanced valve system (14). All common medication for inhalation therapy can be nebulised effectively.

Advanced treatment
This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (13-A, reducing medication loss during exhalation (13-B).

EN

Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.
▶ Mount the membrane on the valve-holder and use the mouthpiece with expiratory valve (13-A).
▶ Follow the usage steps as described in «Section 4.»
☞ Only using the mouthpiece with expiratory valve (7) enables the synchronized functioning of the nebuliser.

Short treatment

The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.
☞ A synchronized nebulisation is not possible using this treatment option.

6. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (6) should be cleaned with a clean, moist cloth.
☞ Always wash your hands well before cleaning and disinfecting the accessories.
☞ Do not expose the compressor or water or heat.
☞ Replace the air hose for each treatment with a new patient or in case of impurities.

7. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact MicroLife-Service (see foreword).
▶ We recommend replacing the nebuliser, mouthpiece, and the face masks after 3 months use.
▶ Check the filter continually for cleanliness and replace it if dirty, or after a maximum of 3 months use. Spare filters are provided with the device.
▶ To replace the filter, open the air filter compartment (4) on the top of the device marked «Filter» and exchange the filter (1).
☞ Never unscrew the plastic lid.

8. Malfunctions and Actions to take

The device cannot be switched on
• Ensure the power lead (2) is correctly plugged into the socket.
• Ensure the ON/OFF switch (3) is in the position «I».
• The device has automatically switched off due to overheating. Wait until it has cooled down and try again.
The nebuliser functions poorly or not at all
• Ensure the air hose (6) is correctly connected at both ends.
• Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.
• Ensure the nebuliser (5) is fully assembled and the colored vapouriser head (13-B) is placed correctly.
• Ensure the required medication has been added.

9. Guarantee

This device is covered by a **3 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.
• The guarantee covers only the compressor. The replaceable components like nebuliser, masks, mouthpiece, air hose, and filters are not included.
• Opening or altering the device invalidates the guarantee.
• The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

10. Technical Specifications

Nebulisation rate: 0.5 ml/min. (NaCl 0.9%)
Particle size: 75% < 5 µm NaCl 0.9%
4.86 µm (MMD with 0.9% NaCl using Malvern system)

Compressor air flow: 13 l/min.
Acoustic noise level: 53 dBA
Power source: 230V~ / 50Hz / 150VA
Power lead length: 1.5 m
Nebuliser capacity: min. 2 ml; max. 12 ml
Residual volume: 0.7 ml
Operating limits: 30 min. On / 30 min. Off
Operating conditions: 10 - 40 °C / 50 - 104 °F
10-95 % relative maximum humidity
Storage conditions: -25 - +70 °C / -13 - +158 °F
10-95 % relative maximum humidity

Weight: 1300 g
Dimensions: 209 x 140 x 124 mm
IP Class: IP20
Reference to standards: EN 13544-1; EN 60601-1; EN 60601-1-2; IEC 60601-1-11; CE0434

Expected service life: 5 years

Declaration of compliance.
Technical alterations reserved.

Date of production: first four digits of the LOT number of the device. First and second digit – the week of the year / third and fourth digit – the year of production.

11. Included accessories

Medical piston compressor, nebuliser, air hose, mouthpiece for inhalation through the mouth, nose piece for inhalation through the nose, adult face mask, child face mask, set of air filters, air valve, instruction manual, guarantee card, carrying bag

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